



Silicon Labs CP210X Driver Installation Menu

Step1. Execute Setup file (CP210x_VCP_Win2K_XP_S2K3.exe)


Step2. Select "Next" button

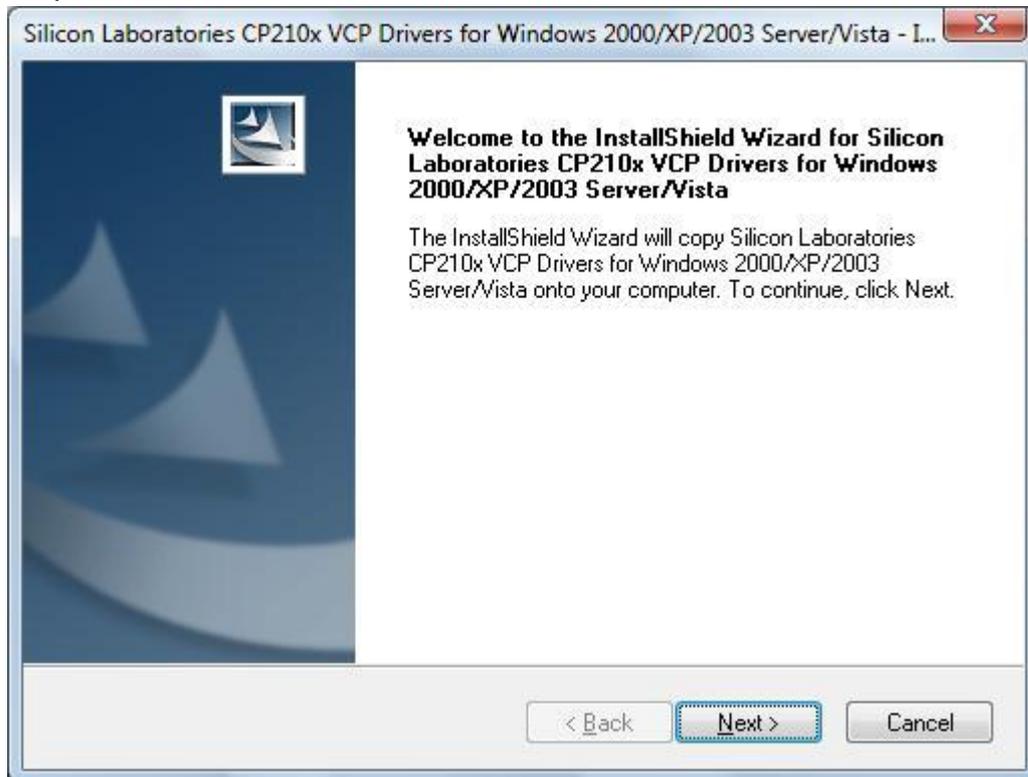


Step3. Accept the license agreement and next

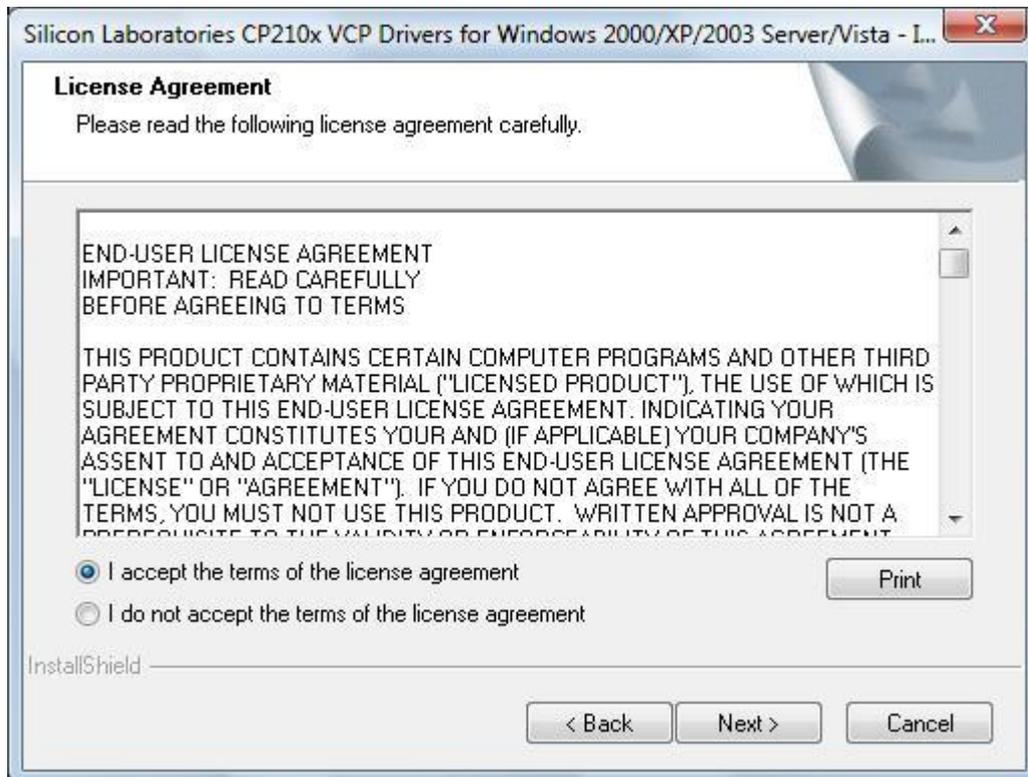


Step4. Choose Driver Destination path

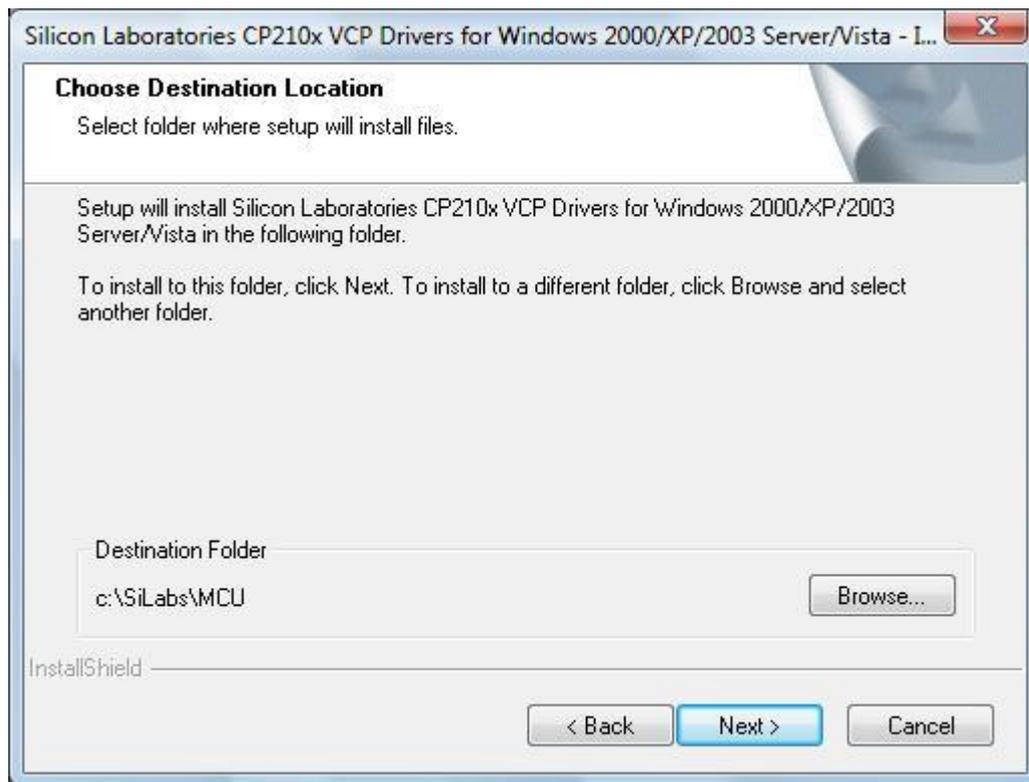


Step5. Confirm Installation , Select "Install" button

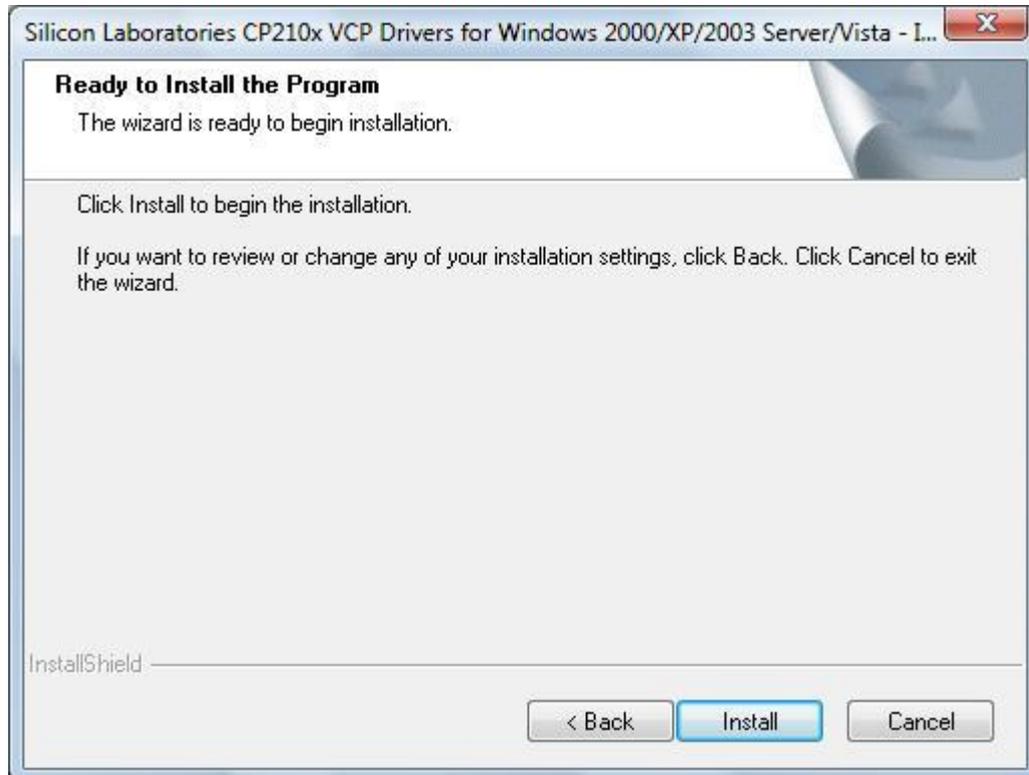


Step6. Driver setup status. If you want to cancel setup, please select "Cancel" button.

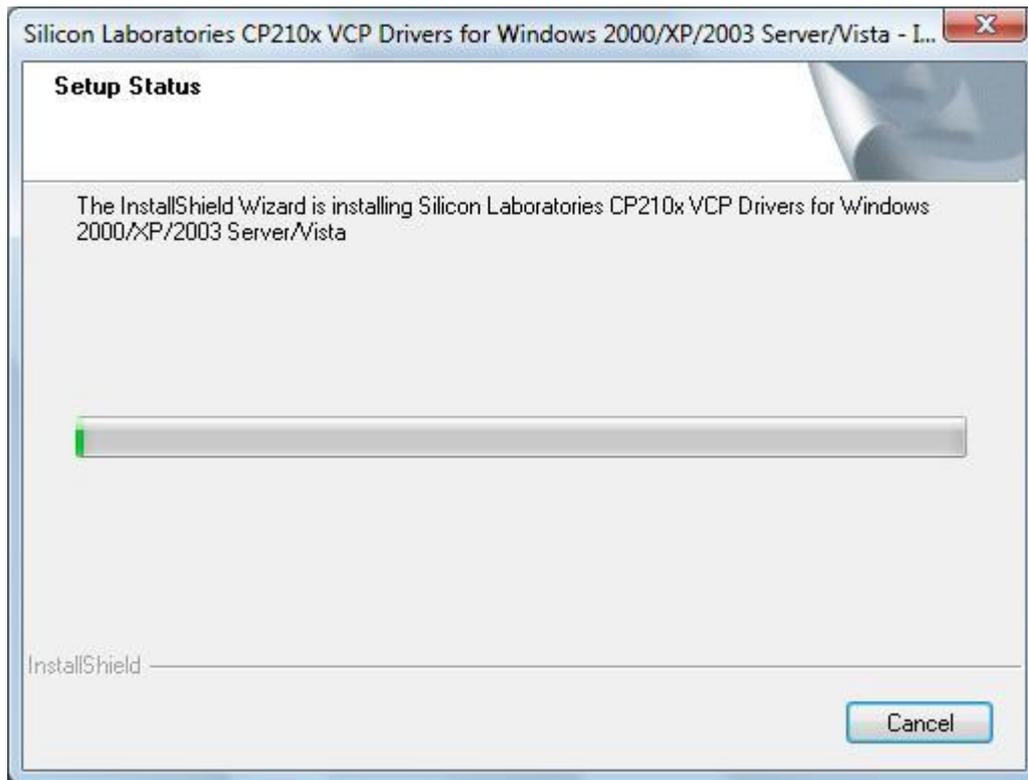


Step7. Launch the CP210x VCP Driverinstaller

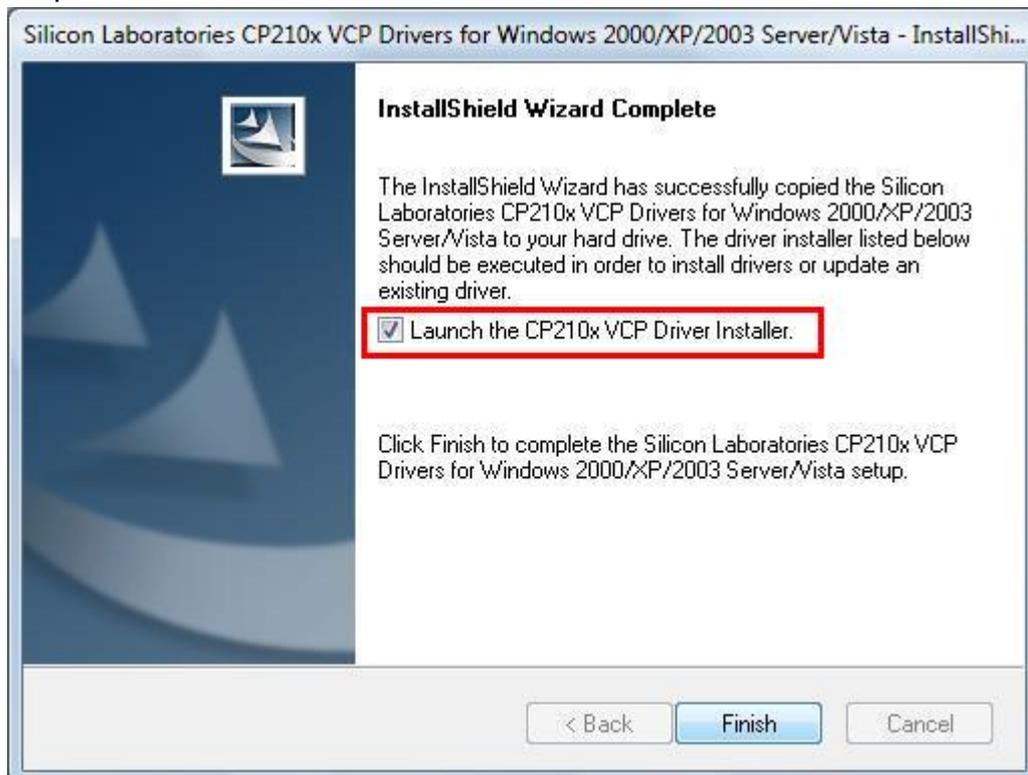

Step8. Select “Install” button

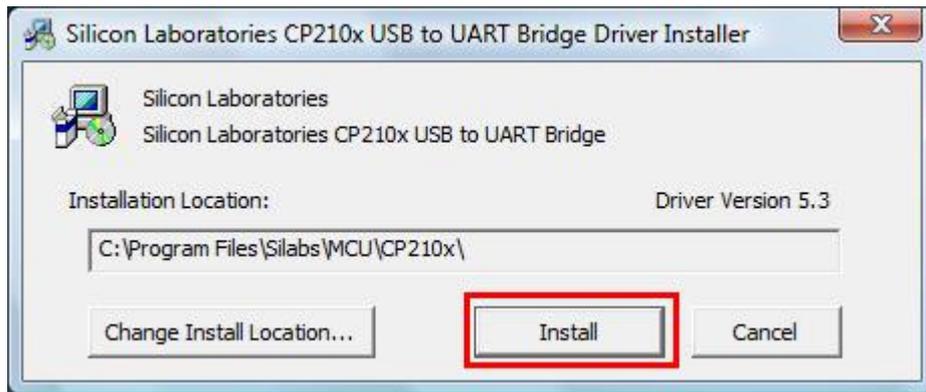

Step9. Installation completed.

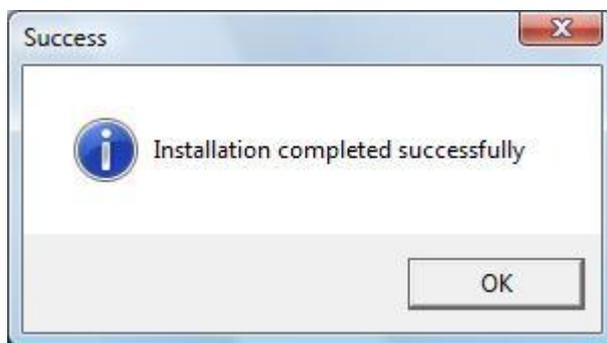

Step10. plug in the usb cable to PC, and the device manager will show:

